


**Alere**<sup>™</sup>

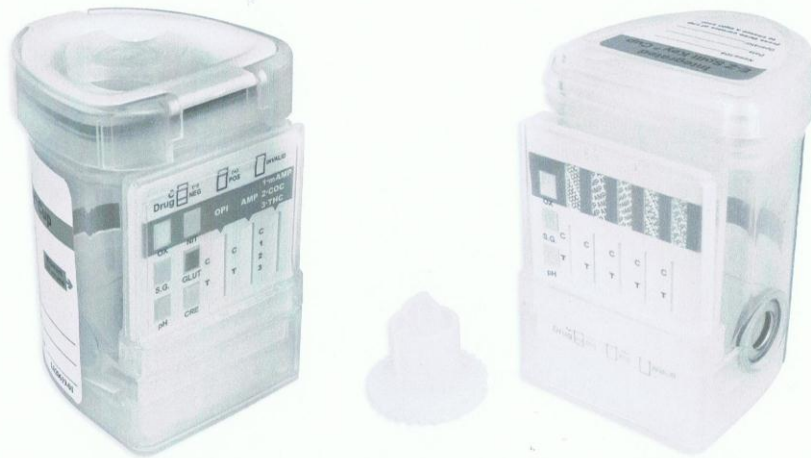

## SureStep<sup>™</sup> Drug Screen Test.

Multi-Drug Test panel with Integrated E-Z Split Key<sup>™</sup> Cup I & Cup II

**A complete drug testing system in a sealed cup.**

The validated secure way to test for drugs in Occupational Medicine and Rehabilitation Centres.

**Alere**<sup>™</sup>

### Multi-Drug Test Panel with Integrated E-Z Split Key<sup>™</sup> Cup I & Cup II.

#### Drugs and cut-offs

Name	Abbreviation	Urine cut off (µg/ml)
Amphetamine	AMP	300*
Barbiturates	BAR	300
Benzodiazepines	BZO	200*
Cocaine	COC	300*
Ecstasy	MDMA	500
Cannabis	THC	50*
Methadone	MTD	300
Methamphetamine	MET	300*
Morphine	MOP	300*
Phencyclidine	PCP	25
Tricyclic Antidepressants	TCA	1000

\*The above cut-offs are compliant with AS/NZS 4308:2008.

#### Instructions for use

1. Collect urine specimen in the cup and secure cap
  2. Check the temperature label (Temp Label) up to 4 minutes after specimen collection
  3. Push the key into the socket of the cup to begin the drug screen test
  4. Read results shown at 5 minutes
- Refer to Instructions For Use provided within kit for full instructions.

#### Result interpretation

A line must appear in the control line region (C) to indicate that the test has worked correctly.

**Please contact Alere for further information.**

#### Alere Australia Pty Ltd

532 Seventeen Mile Rocks Rd  
Sinnamon Park, QLD, 4073  
Australia  
+61 7 3363 7100 Office  
+61 7 3363 7199 Fax  
au.enquiries@alere.com.au Email

[alere.com.au](http://alere.com.au)

#### Alere New Zealand Ltd

PO Box 105653  
Auckland City, Auckland, 1143  
New Zealand  
0800 466 951 Office  
0800 466 958 Fax  
nz.enquiries@alere.com Email

[alere.co.nz](http://alere.co.nz)

#### Kit size, storage and stability

Multi-Drug Test Panel with Cup system. Kit size: 25 tests. Storage: 2–30 degrees °C. Shelf life: about 2 years after date of manufacture.

#### Drug combinations and other SureStep drug tests

The Multi-Drug test panel with Integrated E-Z Split Key<sup>™</sup> Cup I & Cup II formats allows any combination of up to 6 drugs from an extensive menu of 20 different assays. A maximum of 12 assays are available, depending upon the exact combination required. Other formats available within the SureStep<sup>™</sup> brand include: Single and Multi-Drug Test devices, Drug Test Panels, Adulteration Tests and Saliva Tests.

#### Features and benefits

- **Secure:** Eliminates exposure to urine and provides temperature verification to confirm urine samples
- **Convenient:** Simple 3 step procedure – collect the urine, push the key and read results at 5 minutes
- **Flexible:** Drug combinations from 6 drugs to 12 drugs
- **Stable:** Long shelf life (about 2 years after date of manufacture) stored either at room temperature or refrigerated (2–30 degrees °C)
- **Performance:** Accurate and high level of correlation with reference methods GC/MS
- **Verified to AS/NZS 4308:2008 Appendix B (Compliance certificate available)\***

\* Only applicable to 6 panel test.



For more information please contact:

**ADVANCE DIAGNOSTICS NZ LTD**  
0800-2-TESTDRUG - 0800-283-783  
info@advancediagnostics.co.nz  
www.advancediagnostics.co.nz